

WITNESS® | Relaxin

Canine and Feline Pregnancy Test Kit

THE EASY WAY TO DETECT CANINE/FELINE PREGNANCY

EASY TO INTERPRET

Simple-to-understand results

EASY TO USE

Demonstrated:

- Rapid Immuno Migration (RIM™) technology allows for an easy, efficient process.
- Needs only 2 drops of plasma or serum (no whole blood) and 2 drops of buffer.

RAPID RESULTS

Accurate results in approximately
10 minutes

NO REFRIGERATION

Room temperature storage; 18-month
shelf life (*from the date of manufacture*)



**SIMPLY A SMART CHOICE FOR
POINT-OF-CARE DIAGNOSTIC
SOLUTIONS—SO YOUR
CLIENTS CAN REST EASY**

Experience the demonstrated
accuracy, affordability, and
simplicity of WITNESS® RELAXIN
test kits.



Canine and Feline
Pregnancy Test Kit





zoetis

WITNESS[®] RELAXIN

Canine and Feline Pregnancy Test Kit

WITNESS RELAXIN may detect pregnancy as early as 20 days post-LH surge in canines and 20 days after mating in felines. Factors such as breed, size of the canine/feline, and litter size may influence the level of serum relaxin. Although approximately 80% of pregnant canines/felines are detected between 20 and 28 days, some may not be detected as positive until day 31. Negative results should be retested at least seven days later.¹

- Relaxin is secreted primarily by the placenta and is a reliable indicator of pregnancy.
- Levels of relaxin remain elevated throughout gestation and decline quickly following whelping/queening or abortion.

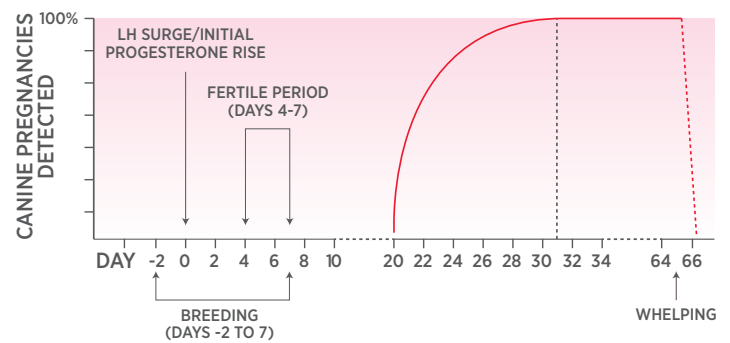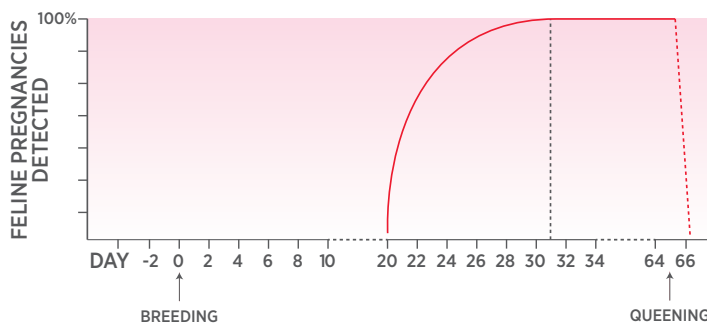

WHAT COULD BE SIMPLER?

1

Apply 2 drops of plasma or serum (no whole blood) using the pipette

2

Add 2 drops of buffer

3

Wait 10 minutes

4

Negative
One line — window 3
Should retest one week later

Positive
Two lines — windows 2 and 3
Retest to confirm positive result

For more information, visit Animalhealthdx.com. To order, call Zoetis customer service at 1-888-ZOETIS-1 (963-8471) or visit shop.zoetisus.com.

References

1. DiGangi BA, Griffin B, Levy JK, Smith BF, Baker HJ. Use of a commercially available relaxin test for detection of pregnancy in cats. *J Am Vet Med Assoc.* 2010;237:1267-1274.

All trademarks are the property of Zoetis Inc., its affiliates and/or its licensors. ©2015 Zoetis Inc. All rights reserved. September 2015. WIT-00100

DETECT. PREVENT. TREAT.

zoetis